



EVERYTHING YOU WANT TO KNOW AT A GLANCE


With the new pearly® Cycle app

Easily synchronize your pearly® with your smartphone via Bluetooth and you can view your current fertility status and your cycle information of the last 99 days. The app shows you everything you need to get a good overview. pearly® Cycle is suitable for all women who like to concentrate on the essentials.

With pearly® you can discover your own individual cycle. pearly® is the newest cycle computer from VE Valley Electronics GmbH, which evaluates your unique menstrual cycle and shows you when you are fertile and when not.

Functions


- Daily and weekly fertility status
- Past cycle data
- Temperature curve of the last 99 days

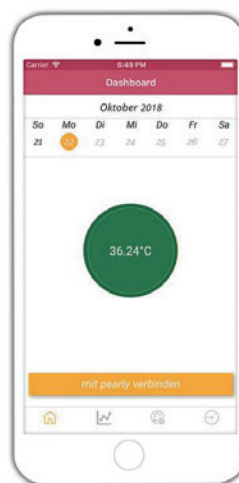

INSTRUCTIONS


Setup and Synchronization

1

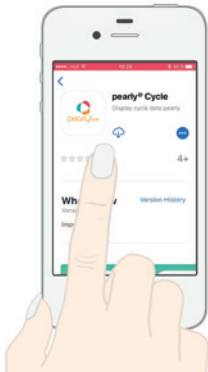


Download pearly® Cycle.

2

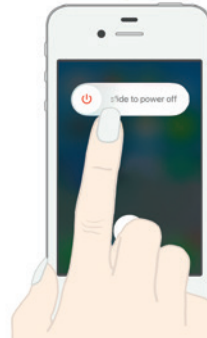


Restart your smartphone.

3

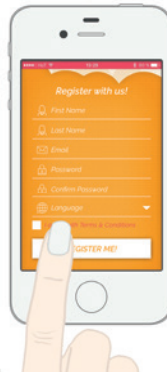


Open pearly® Cycle and register or login to your existing account.

4

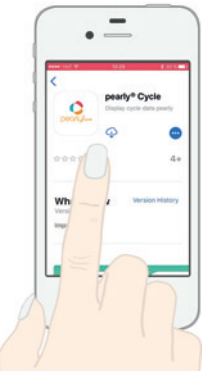

If Bluetooth is not yet activated, change the function status in the settings.

5

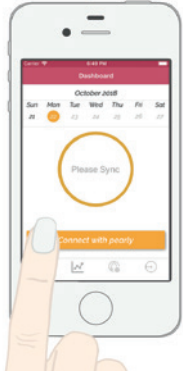

Connect app with pearly®.

6



Switch on pearly® and activate Bluetooth:
Press 2 x circle key
Press 1 x plus key